



ULTRA HILO


OUR PURE WATER,
YOUR MIRACLE

 COSMODERMA




ULTRAPURE

COSMODERMA uses an ULTRAPURE grade of hyaluronic acid. We guarantee to our customers the purity and safety of this product. Because we insist on using only hyaluronic acid from HTL in France.



FORMULA OF ULTRAHILO

HI

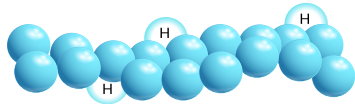

32mg of high molecular weight
(1100~1400 kDa)




LO



32mg of low molecular weight
(80~100kDa)

Thermal unique process with Non-Crosslinked agents used

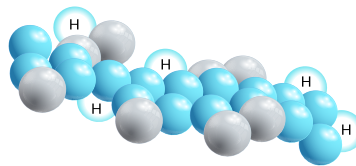

ULTRAHILO





FEATURE



HIGH HA CONCENTRATION (64mg/2ml)³

- Highly effective and efficient
- Extensive Spreadability
- Low viscosity

NO BDDE

- Non-crosslinked product
- No other chemical ingredients





ULTRAHILO UNDER YOUR SKIN

Nothing is softer or more flexible than water,
yet not even your skin can resist it...



Strong skin absorption ability



Smooth flow through the skin



Provide immediate radiance



Increases elasticity and
reduces wrinkles by remodeling
the extracellular matrix




Improvement in skin
density and skin tone



Maintaining best conditions for
the viability of fibroblast,
keratinocytes and adipocytes
(See proliferation test)

PROLIFERATION TEST

HaCaT Proliferation Test

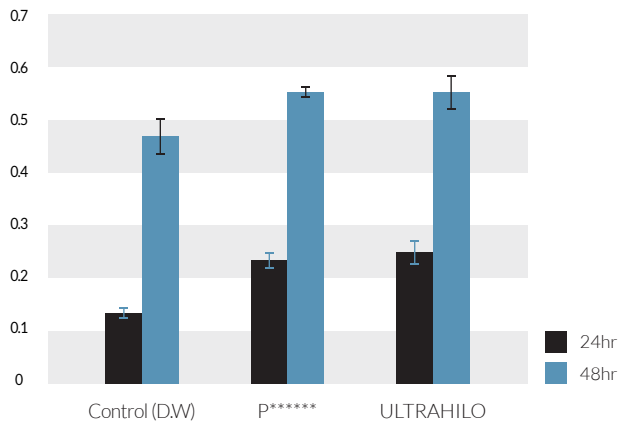


SAMPLE	24h(%)	48h(%)
Control (D.W)	100	342
P*****	167	410
ULTRAHILO	183	412

HaCaT: Human keratinocyte

Overall the test result of Ultrahilo is up to 10% higher than P*****. Maintaining best conditions for the viability of fibroblast, keratinocytes and adipocytes.

NIH3T3 Proliferation Test

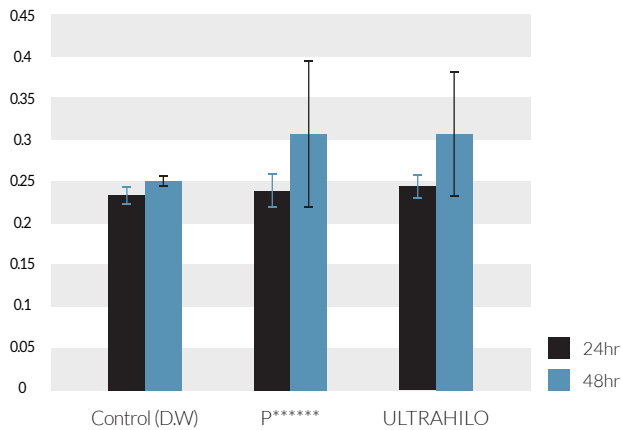

SAMPLE	24h(%)	48h(%)
Control (D.W)	100	108
P*****	103	130
ULTRAHILO	104	132

NIH3T3: Mouse fibroblast

Overall the test result of Ultrahilo is same of slightly higher than P*****.

INTENDED USE

Anti aging and wrinkle removal > Elastic fibers and collagen
Acne and scars > Dermal cell regeneration

ULTROHILO can also recover compromised adipose tissue.



RESULT



Face treatment

Quantity | 1ml per side
Treatment sessions | 2 treatments
(1month interval)
Frequency | Twice per year




Neck treatment

Quantity | 2ml
Treatment sessions | 2 treatments
(1month interval)
Frequency | Twice per year



Inner arms treatment

Quantity | 2ml per arms
Treatment sessions | 2 treatments
(1month interval)
Frequency | Twice per year

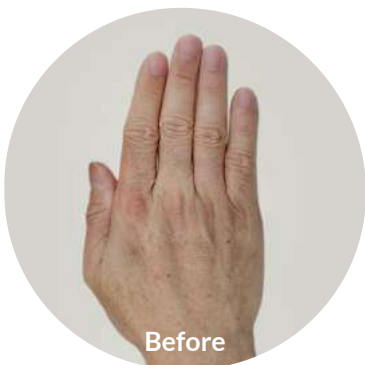

Hand treatment

Quantity | 2ml
Treatment sessions | 2 treatments
(1month interval)
Frequency | Twice per year

ULTRAHILO

OUR PURE WATER,
YOUR MIRACLE

 COSMODERMA



 COSMODERMA

Usage of ULTRAHILO




 **ULTRAHILO**
3.2% - 32mg(H-HA) + 52mg(L-HA) / 2ml
Hyaluronic acid sodium salt
2ml pre-filled syringe



+ FACE

Technique	BAB 29G 13mm needle
Product quantity	1ml per side
Treatment sessions	2 treatments
Treatment interval	4 weeks
Frequency	Twice per year

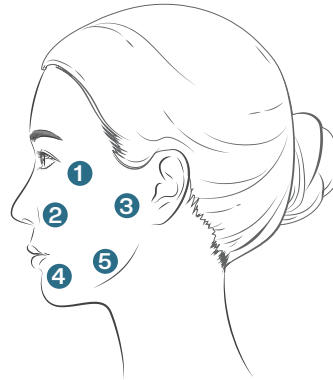

DETAIL

The 5FP TECHNIQUE (5 Face Point)

1 ZYGOMATIC PROTRUSION

At least 2cm away from the external corner of the eye



2 NASAL BASE

- Draw a line connecting the nostril and tragus
- Draw a perpendicular line starting from the pupil
- Locate the injection point at the intersection of the 2 lines



3 TRAGUS

1cm anterior to the bottom of the tragus



4 CHIN

- Draw a vertical line in the center of the chin
- Draw a perpendicular line one third from the top of the vertical line
- From the point of intersection move 1.5cm towards the oral commissures




5 MANDIBULAR ANGLE

1cm above the mandibular angle




+ NECK

Technique	1 bolus + fanning 29G 13mm needle 25G 50mm cannula
Product quantity	2ml per side
Treatment sessions	2 treatments
Treatment interval	4 weeks
Frequency	Twice per year

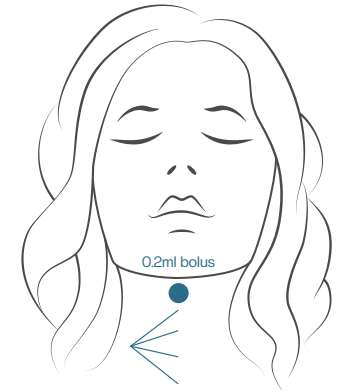

+ HAND

Technique	Fanning 25G 50mm cannula
Product quantity	2ml per side
Treatment sessions	2 treatments
Treatment interval	4 weeks
Frequency	Twice per year

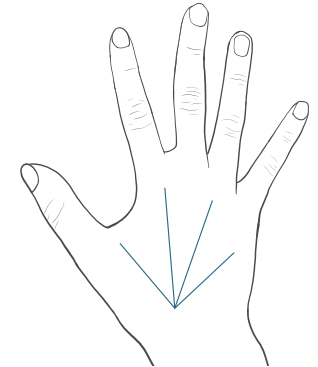

+ INNER ARM

Technique	7 point technique 29G 13mm needle
Product quantity	2ml per side (0.2-0.3ml / bolus)
Treatment sessions	2 treatments
Treatment interval	3 weeks
Frequency	2-3 times per year

